


FUNCTIONS

The e-Dx instrument is the next generation in endodontic diagnostic instruments. This unique instrument can easily cold and fracture test teeth. Using our special open cell foam insert, the e-Dx instrument stores dental coolants longer enabling a doctor to cold test multiple teeth. The detachable contra-angled pliers are designed to securely hold the foam insert and reach posterior teeth to provide improved visibility during cold testing. The built in FracTester™ can be placed onto a tooth cusp or other suspected areas to help detect fractures.

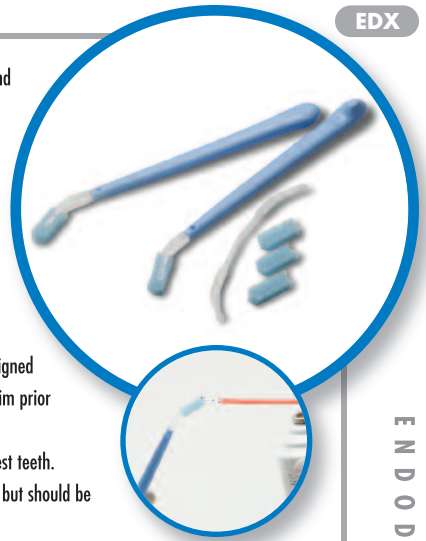

TECHNICAL SUMMARY

e-Dx HANDLE:

The e-Dx handle is constructed of a blue colored special resin that is durable and yet can be used comfortably to fracture test teeth. The handle can be reused after processing in a hospital grade disinfectant with a tuberculocidal claim.

e-Dx PLIER TIPS:

The e-Dx pliers tip is molded of a semi transparent thermoplastic resin. The tip reversibly mounts to the handle and is designed to engage and firmly grip the foam tip. The tips can be processed in a hospital grade disinfectant with a tuberculocidal claim prior to use, but are designed to be **single-use**.*

e-Dx FOAM INSERTS:

Foam tips are manufactured from unique open cell foam that will contain several brands of dental coolants used to cold test teeth. It is designed to be specifically held by the e-Dx handle and is blue in color. They may be autoclave processed before use, but should be properly disposed of after single-use.*

PACKAGING

All e-Dx kits and refills are packaged in poly bags and bar coded to HIBC convention.

PRODUCT CODES REF# ITEM DESCRIPTION

- EDX-s** e-Dx Kit: 3 e-Dx handles, 36 plier tips and 36 foam inserts
- EDXRP-s** e-Dx Refill Pack: 36 plier tips and 36 foam inserts
- EDX** Case of e-Dx packs: 12 per case
- EDXRP** Case of e-Dx refill packs: 12 per case

COMPARISON GUIDE

FEATURES	e-DX	TOOTH SLOOTH™	COTTON APPLICATOR STICKS	COTTON BALLS
Multi use instrument	YES	NO	NO	NO
Cold tests several teeth	YES	NO	NO	LIMITED—difficult to use
Controlled fracture testing	YES	YES	NO	NO
Single use cold testing	YES	NO	NO	YES—difficult to use
Unique foam easily holds coolants	YES	NO	NO	NO



800/752.2812
(Inside the USA)

503/531.3904
(Outside the USA)

503/531.3757 FAX

www.jordco.com

FEATURES & BENEFITS

- Dual Diagnostic Instrument** Single instrument that cold and fracture tests teeth.
- Contra Angled Pliers** The contra angled design reaches posterior teeth with improved visibility.
- FracTester™ Handle** The FracTester's pyramidal bite feature is placed onto cusp tips or other suspect areas to help detect fractures.
- Single-Use Pliers and Foam Insert** Single-use components help to reduce cross contamination.
- Cold Test Multiple Teeth** To save valuable chair time, open cell foam holds extra coolant that prolongs use and can be recharged as needed.
- Use Current Coolant Spray** No special coolant required. Many coolants are available through your dental dealer.*
- Reusable Handle** The FracTester™ handle can be reprocessed using a hospital grade disinfectant with a tuberculocidal claim. To save time after processing, your assistant can preload a new pliers and foam tip onto the handle and store for future use.
- Snap Assembly** The unlocked pliers tip inserts into the FracTester handle with a positive snap. After use, the pliers and foam can be easily removed and disposed of properly.
- Foam Locks Into Pliers** The foam tip locks securely into the pliers tips for precise control and to minimize accidental aspiration of the foam tip.
- Precise Cold Testing** The unique foam can be trimmed as needed to precisely fit the tooth surface and helps prevent accidental cold exposure to soft tissue.

*Recommended Coolants: Endo Ice® by Hygienic Premier; Parmaethyl by Sepodont; Artic Blast by Marie Royco



The blue color of the ENDORING E-FOAM insert is a trademark registered in the U.S. and is owned by Jordco, Incorporated, Beaverton, Oregon U.S.A. The trademark ENDORING, E-FOAM and the E-FOAM SHAPE is a trademark registered in the U.S. and in the European Community, and they are owned by Jordco, Incorporated, Beaverton, Oregon, U.S.A. JORDCO is a trademark registered in the United States, Canada and in Europe and it is owned by Jordco, Incorporated, Beaverton, Oregon U.S.A.

* 	② SINGLE USE	FOAM INSERTS AND PLIERS: Single use only. Properly dispose of these items after use. HANDLES: May be reprocessed using a hospital grade disinfectant with tuberculocidal claim. WARNING: Discard the handle if the foam plier can no longer be safely retained.	U.S. Patent Pending Made in USA	
-------	-----------------	---	------------------------------------	--