

ENDORING® FILECADDY®

EFC

FUNCTIONS

The Endoring FileCaddy organizer provides a unique bulk storage solution for all endodontic files. The clinician places multiple files securely into the special foam insert, closes the lid and steam autoclaves the entire FileCaddy organizer—no bag or wrap required. Single handed operation allows easy opening of the protective lid to access the processed files at chair side. Files are easily identified, selected and transferred to the companion Jordco Endoring II organizer for the current endodontic procedure. The FileCaddy organizer disassembles easily to allow replacement of the foam insert.

TECHNICAL SUMMARY

FILECADDY CASE:

The Endoring FileCaddy case is constructed of a special glass reinforced thermoplastic resin that can be safely steam autoclaved (275°F/135°C). There are three molded components that make up the FileCaddy case; a molded base, retainer and lid. All the components are engineered to easily and reliably assemble. A specially designed retainer locking mechanism securely holds the foam insert onto the base. The color of the case is a light beige tone. Size (assembled FileCaddy): Approximately 4 5/8" w x 4 5/8" d x 3" h.

FILECADDY FOAM INSERT:


The disposable EFC Foam Insert is constructed of Jordco's proprietary open cell foam that is also used in our Jordco e-Foam® Inserts. The EFC Foam Insert and case can be steam autoclave processed (275°F/135°C). The foam inserts are available in blue or yellow color.

PACKAGING

The Endoring FileCaddy organizer is packaged in a corrugated carton with one Foam Insert Combo package (6 yellow and 6 blue). Foam Inserts are packaged in a clear poly bag.

PRODUCT CODES

REF#	ITEM DESCRIPTION
EFC-s	Single Endoring FileCaddy Kit: 1 Endoring FileCaddy assembly and 1 FileCaddy Combo Foam Inserts pack (6 blue and 6 yellow)
EFCF-s	Single Endoring FileCaddy Combo Foam Inserts: Single package of 12 inserts (6 blue and 6 yellow)
EFCFB-s	Single Endoring FileCaddy Blue Foam Inserts: Single package of 12 inserts (blue only)
EFCFY-s	Single Endoring FileCaddy Yellow Foam Inserts: Single package of 12 inserts (yellow only)
EFC	Case of Endoring FileCaddy Kits: 12 kits per case
EFCF	Case of Endoring FileCaddy Foam Inserts Combo: 24 single packs per case (12 blue and 12 yellow)
EFCFB	Case of Endoring FileCaddy Foam Inserts Blue: 24 single packs per case (blue only)
EFCFY	Case of Endoring FileCaddy Foam Inserts Yellow: 24 single packs per case (yellow only)

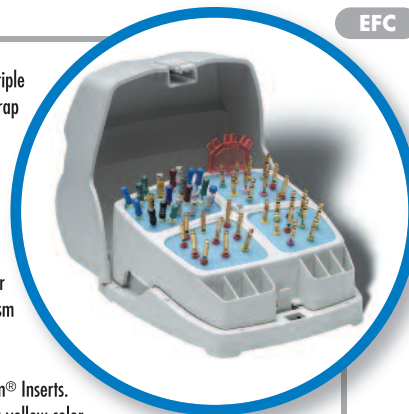

ENDODONTIC FILE STORAGE DEVICE

COMPARISON GUIDE

FEATURES	FILECADDY	ENDO CLEAN STAND (DENTSPLY)	METAL ENDO BOXES (VARIOUS MFG.)
Replaceable foam insert	YES	YES	NO
Attached locking lid	YES	NO	NO (HAS LID)
4 bulk storage compartments	YES	SINGLE	MULTI SINGLE HOLE
One-hand operation	YES	NO	NO
Use at chair side	YES	NO	YES
Steam Autoclavable	YES	YES	YES
Spill proof foam insert	YES	YES	NO
Autoclave bag required	NO	NO	NO



800/752.2812
(Inside the USA)

503/531.3904
(Outside the USA)

503/531.3757 FAX

www.jordco.com

FEATURES & BENEFITS

Locking Lid Files are held securely in place between procedures and will not spill out if the FileCaddy organizer is dropped or turned upside down.

Autoclavable and Durable The FileCaddy organizer is constructed of a medical grade resin that can withstand repeated steam autoclaving (275°F/135°C). The resin is glass reinforced for added durability and longevity.

4 Bulk Storage Compartments The FileCaddy organizer is divided into 4 convenient bulk storage compartments to help make file organization easy. The separate compartments simplify organization and retrieval of instruments. Over 100 instruments can be safely stored in the FileCaddy organizer for steam autoclaving.

Disposable Foam Inserts The foam inserts are disposable to reduce cross contamination*.

Two Colors of Foam Inserts Two colors are helpful to organize/color code file setups. Inserts can be purchased in three different refill packs: Combo (6 Blue/6Yellow), Blue (12 blue) or Yellow (12 yellow) packs.

Secure File Loading Foam inserts eliminates the risk of spilling files – minimizes lost files.

One-Hand Operation One hand operation allows for easy opening of the locking lid to access files and contents.

Simple to Use Bulk load endodontic files, close locking lid and steam autoclave.

Save Files Steam autoclave only those files that are actually used in the procedure. No unnecessary steam autoclaving of unused files.

* The Foam Insert should be replaced every two autoclaving cycles to prevent discoloration of the EFC resin case. Should discoloration occur, simply disassemble the EFC and soak the components in sodium hypochlorite over night to remove discoloration.

The blue color of the ENDORING E-FOAM insert is a trademark registered in the U.S. and is owned by Jordco, Incorporated, Beaverton, Oregon U.S.A. The trademark ENDORING, E-FOAM and the E-FOAM SHAPE is a trademark registered in the U.S. and in the European Community, and they are owned by Jordco, Incorporated, Beaverton, Oregon, U.S.A. JORDCO is a trademark registered in the United States, Canada and in Europe and it is owned by Jordco, Incorporated, Beaverton, Oregon U.S.A.



Steam autoclave only prior to use. **DO NOT REUSE FOAM INSERT.** Autoclaving **will not** remove bioburden left behind from prior usage. Do not exceed temperature of **275°F (135°C)**.
Warning: Do not place any components into Surgical Milk or instrument dips prior to autoclaving.

U.S. Patents Pending
Made in USA

