

ENDORING® II & ENDORING GELWELLS®

ER2

FUNCTIONS

The Endoring II organizer enables the clinician to directly place, store, measure and clean endodontic hand and rotary instruments within the operating field. It reduces the risk of cross-contamination and minimizes the passing of sharps between dentists and staff. Using the Endoring II organizer helps make endodontic procedures safer and faster.

TECHNICAL SUMMARY

ENDORING II:

Assemblies are molded from an autoclavable blue thermoplastic resin. Assembly consists of six pieces: foam-holder cup, reusable Gelwell cup attachment, disposable Gelwell cup holder attachment, large, medium and small snap-on finger rings. Note: the foam-holder cup has a molded plastic 30-mm scale for right and left handed users.

ENDORING METAL RULER:

Autoclavable anodized blue aluminum detachable ruler with contrasting white laser-etched scales in 1mm and 1/2mm increments. Accessory scale provides marking up to 37mm for extra long canals. Ruler can be used by right and left handed users. Available with or without rubber stop locks.

GELWELL SYSTEM:

Reusable Cup: 2ml cup is molded from an autoclavable blue thermoplastic resin and attaches to the Endoring II organizer. Disposable Cups: Single-use 0.65ml cups molded from a clear thermoplastic resin. Disposable cup is placed into the holder and attaches to the Endoring II organizer.

e-FOAM®:

Single-use* open cell yellow and blue foam. Can be steam autoclave processed prior to use. During steam autoclave processing, endodontic instruments can be inserted into the foam.

PACKAGING

All Endoring kits, packs and inserts are packaged in poly bags and bar coded to HIBC convention.

PRODUCT CODES


REF# ITEM DESCRIPTION

- ERK2-s** **Single Endoring II Premium Kit:** 1 Endoring II assembly (blue), 1 standard metal ruler, 1 e-Foam insert combo pack (48 inserts), 1 pack Gelwells (48 single-use cups) and 1 EndoGel pack (12 1ml MicroDose tubes—USA kits only)
- ER2SK-s** **Single Endoring II Starter Kit:** 1 Endoring II assembly (blue), 1 e-Foam insert combo pack (48 inserts) and 2 single-use Gelwells
- ERR-s** **Single Endoring Metal Ruler:** Standard with stop locks
- ERRFL-s** **Single Endoring Metal Ruler:** Left handed no-stop locks
- ERRFR-s** **Single Endoring Metal Ruler:** Right handed no-stop locks
- ERG-s** **Single package of 48 single-use Gelwells**
- ERF-s** **Jordco e-Foam Combo Pack:** Single package of 48 foam inserts (24 blue and 24 yellow)
- ERFB-s** **Jordco e-Foam Blue Pack:** Single package of 48 foam inserts
- ERFY-s** **Jordco e-Foam Yellow Pack:** Single package of 48 foam inserts

Dealers Only: All products are available for discounted case pricing, please call for details.

REF# ITEM DESCRIPTION

- ER2-s** **Single Endoring II with metal ruler:** 1 Endoring II assembly (blue), 1 standard metal ruler, 6 e-Foam inserts, and 4 single-use Gelwells
- ER2G-s** **Endoring II with metal ruler:** 1 Endoring II assembly (green), 1 standard metal ruler, 6 e-Foam inserts, and 4 single-use Gelwells
- ER2Y-s** **Endoring II with metal ruler:** 1 Endoring II assembly (yellow), 1 metal ruler standard, 6 e-Foam inserts, and 4 single-use Gelwells
- ER2V-s** **Endoring II with metal ruler:** 1 Endoring II assembly (violet), 1 standard metal ruler, 6 e-Foam inserts, and 4 single-use Gelwells
- ER2BX-s** **Endoring II without metal ruler:** 1 Endoring II assembly (blue)
- ER2GX-s** **Endoring II without metal ruler:** 1 Endoring II assembly (green)
- ER2YX-s** **Endoring II without metal ruler:** 1 Endoring II assembly (yellow)
- ER2VX-s** **Endoring II without metal ruler:** 1 Endoring II assembly (violet)
- ER2CVP-s** **Endoring II Color Value Pack:** 3 Endoring II assemblies one each yellow, green and violet (metal ruler not included)

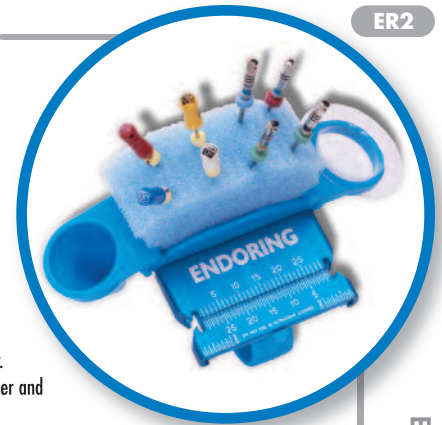

ENDODONTIC HAND INSTRUMENT

COMPARISON GUIDE

FEATURES	ENDORING II	ENDORULERS	ENDO-CUSHIONS	MEASURING GAUGE
Autoclavable	YES	YES/NO	NO	YES
Reduces instrument sticks	YES	NO	NO	NO
Disposable foam inserts	YES	NO	YES	NO
Cleans files during use	YES	NO	YES	NO
Reusable or disposable lubricating gel holder	YES	NO	NO	NO
Finger mounted	YES	YES	NO	NO
Direct operator control of all files	YES	NO	NO	NO
Right-left 30mm scale	YES	NO	NO	NO
Easy calibration of curved instruments	YES	NO	NO	NO
Faceted foam surface to separate files	YES	NO	NO	NO
Detachable anodized laser etched ruler	YES	NO	NO	NO



800/752.2812
(Inside the USA)

503/531.3904
(Outside the USA)

503/531.3757 FAX

www.jordco.com

FEATURES & BENEFITS

- Hand-held Instrument** Allows direct operator control of hand and rotary instruments. Eliminates time consuming passing of dangerous files from assistant or bracket tray. Minimizes eyestrain and loss of attention from searching for files, etc. Saves several minutes each procedure.
- Held on Index Finger** Frees both hands. Allows operator to hold mouth mirror in one hand while selecting and operating files or instruments with the opposite hand. Includes three ring sizes (large, medium and small) to fit a majority of fingers. The small ring is ideal for the petite hand.
- Two Foam Color** Jordco blue and yellow e-Foam inserts provide an additional option for color coding instrument set-ups.
- Standard Right-Left 30mm Scale** The built in Endoring ruler has a recessed calibration tract and improved rubber stop locks. These features enable the operator to position files, reamers, paper points, gutta percha and other endodontic instruments into the tract and rapidly calibrate them to working length. The scale is designed for both right and left handed users. (Also available with or without stop locks and groove)
- Detachable Laser Etched Metal Ruler** Autoclavable metal ruler with a precise laser etched right or left-handed 30mm scale. Includes 1/2 mm calibration markings on the lower half of the scale. The metal ruler also includes a secondary scale to measure files up to 37-mm in length. Includes a calibration tract for easy positioning of instruments. Ruler can be easily detached for cleaning and processing.
- Easy Calibration of Curved Instruments** Curved files can be easily positioned into the calibrating tract within the standard or metal ruler and temporarily straightened with slight pressure. After the stop is adjusted, the file is removed and allowed to reassume its curved shape.

The blue color of the ENDORING E-FOAM insert is a trademark registered in the U.S. and is owned by Jordco, Incorporated, Beaverton, Oregon U.S.A. The trademark ENDORING, E-FOAM and the E-FOAM SHAPE is a trademark registered in the U.S. and in the European Community, and they are owned by Jordco, Incorporated, Beaverton, Oregon, U.S.A. JORDCO is a trademark registered in the United States, Canada and in Europe and it is owned by Jordco, Incorporated, Beaverton, Oregon U.S.A.



Steam autoclave only prior to use. **DO NOT REUSE FOAM INSERT.** Autoclaving **will not** remove bioburden left behind from prior usage. Do not exceed temperature of **275°F (135°C)**.
Warning: Do not place any components into Surgical Milk or instrument dips prior to autoclaving.

U.S. Patents 5368482, 6036490, Patent Pending
Made in USA

