

ENDORING® FILECADDY™

Instructions for Use

Please read before using the Endoring FileCaddy



Jordco, Incorporated

Beaverton, Oregon USA 97006

503/531-3904 TEL, 503/531-3757 FAX, 800/752-2812 toll free USA

www.jordco.com

REF # EFC-s



ENDORING FILECADDY

1




The 3-part Endoring FileCaddy (EFC-s) consists of a base, retainer and lid. The complete kit also contains one Endoring FileCaddy Foam Refill Pack (EFCF-s) that consists of 6 pieces of blue and 6 pieces of yellow foam. All blue and all yellow foam refill packs are also available.

2



Begin assembly by removing a piece of the die cut foam from the perforated pad and loading it onto the FileCaddy base.

3

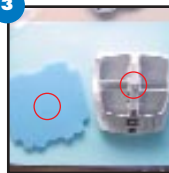


Align the foam refill with the two "V" cuts facing the front of the FileCaddy. Note the center of the foam insert has a small die cut cross that fits onto the cross on the base.

4



Gently press the foam insert onto the cross on the base making sure the two "V" notches are facing the front of the base.

5

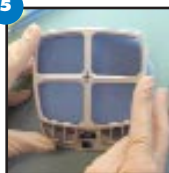

Slide the retainer over the foam insert and engage the snap on the front of the base. Assist seating by slightly pulling back on the snap arm on the base and allowing the retainer to fully "snap" into place.

6




Note that the base snap arm can be engaged with a finger to clear the "catch" on the retainer. When the foam needs to be replaced simply reverse this operation.

7




Place the lid onto the assembled base/retainer by holding the cap in the full open (vertical) position and then rocking the cap onto the base pivot pins as shown in Figure 7. The lid can be easily removed, if desired, during a procedure.

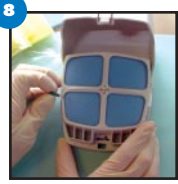


Files and instruments can be securely loaded into the foam insert without concern for them falling out if the FileCaddy is accidentally tipped. When loading, ensure that the files are not placed through the vent holes located on the bottom of the base.



The FileCaddy is ideal for bulk storage of endodontic files or for individual patient procedures. **BULK STORAGE:** Each compartment helps segregate and denote different types of files. Note that the set up in Figure 9 uses a number of rotary and hand files that are all 21mm in length. The FileCaddy securely holds up to 100 files using all 4 compartments. After autoclave processing, clean files can be removed from the FileCaddy before or during a procedure **INDIVIDUAL PATIENT SETUPS:** The FileCaddy can be loaded with a complete patient setup to be used one patient at a time. The foam insert should be changed with each set up.

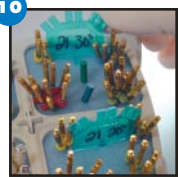

Jordco's Endoring Marking Tabs (EMT-s) can be used to note information such as file size and type. EMTs have a break-away numbering tab that can be used to track the number of times a set of files is used.

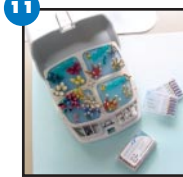

A loaded file caddy with open lid demonstrates the ease with which instruments can be accessed for or during a procedure.



The "accessory bins" located at the front of the retainer can be used to store items such as long shank burs and cotton pellets.



The FileCaddy lid is closed with a simple "click" that indicates that the contents are securely locked in place.



An optional "dust protector" is found with each foam insert pad. Once separated, it can be used to screen the ventilation hole found on the bottom of the base.



Use only the instruments you need! The FileCaddy can be covered with a disposable protector sheet to allow access to instruments during procedures and help reduce cross contamination.



The fully loaded FileCaddy can be processed in steam autoclaves. (275°F/135°C)



To open the FileCaddy, simply press the lid's locking lever arm and rotate the lid into the open position.