

# ENDO GEL®

## Instructions for Use

Please read before using the EndoGel



**Jordco, Incorporated**

Beaverton, Oregon USA 97006


503/531-3904 TEL, 503/531-3757 FAX, 800/752-2812 toll free USA

[www.jordco.com](http://www.jordco.com)

REF# EG-s




1

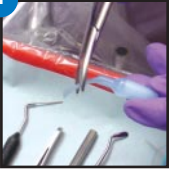


Open the EndoGel microtube by cutting off the tip. If microtube is damaged, do not use.

2

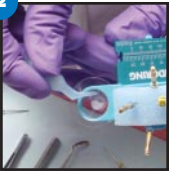


Express EndoGel directly onto a hand held dental file or instrument. The file or instrument can also be coated by dipping into an Endoring II Gelwell cup filled with EndoGel. The file or instrument can then be passed directly into the tooth.

3

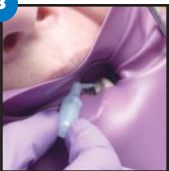

EndoGel can also be expressed from the microtube without pressure into the root canal chamber of the tooth. Always confine the EndoGel within the root canal chamber. Avoid expressing through the apex of the root.

4

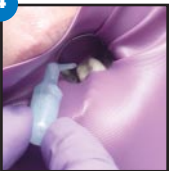

Always isolate the tooth with a sealed rubber dam prior to treatment. If skin or gingival contact with EndoGel occurs, flush with water. If eye contact accidentally occurs, flush with water immediately.

**Note:** Irrigate with sodium hypochlorite to flush the water soluble EndoGel out of the canals and the chamber of the tooth. Oxygen bubbles are generated that will aid in the cleansing of the canals and chamber. Properly dispose of unused contents of microtube after use.