

EMT—ENDORING® MARKING TABS

Instructions for Use

Please read before using the Endoring Marking Tabs


**Jordco, Incorporated**

Beaverton, Oregon USA 97006


503/531-3904 TEL, 503/531-3757 FAX, 800/752-2812 toll free USA

www.jordco.com

The Endoring Marking Tabs are supplied in 3 different colors to aid in organizing file set-ups. 8 EMTs are supplied on a convenient wheel for easy dispensing.

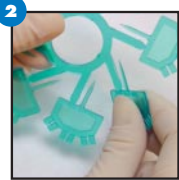

To remove an EMT, gently twist it off the wheel.

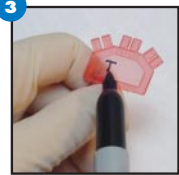

Each EMT has a writing surface for information such as file types or patient name. It is best to use an indelible pen such as a Sharpie to withstand autoclave processing.



Each EMT has 6 numbered snap away tabs located on the top. By using an instrument such as a hemostat, the tabs can be easily removed to identify the number of times a file set-up has been used.



The EMT is easily inserted into a foam insert prior to the next use.



The entire file set-up can be placed in an autoclave bag and be processed for the next use. Each EMT can withstand autoclaving up to 6 times.