

PURE BOND® DISPENSERS

Instructions for Use

Please read before using the Pure Bond Dispensers



Jordco, Incorporated

Beaverton, Oregon USA 97006

503/531-3904 TEL, 503/531-3757 FAX, 800/752-2812 toll free USA

www.jordco.com



1




With the Pure Bond Dispenser lid open, bonding agents can be conveniently placed into any of five wells. Bonding agents can be pre-loaded prior to each procedure helping reduce contamination of bonding agent syringes and small bottles.

2



Avoid cross contamination of syringes by dispensing etchants directly into the well.

3

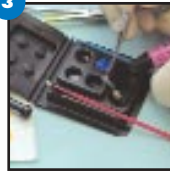

The larger utility well is ideal for holding composites, cotton balls or sponges. Micro tip brushes can be stored in the channels located to the side of the wells.

4



After expensive light sensitive bonding agents have been loaded, the lid can be gently closed to protect them from light contamination until use.

5



Save valuable time by pre-loading several Pure Bond Dispensers early in the day prior to any bonding appointments.

6



During the procedure, the lid is opened and the assistant can pass the Pure Bond Dispenser to the doctor for use. The lid can be closed between applications.