



**ENGLISH**


**PURE BOND DISPENSER INSTRUCTIONS FOR USE:**

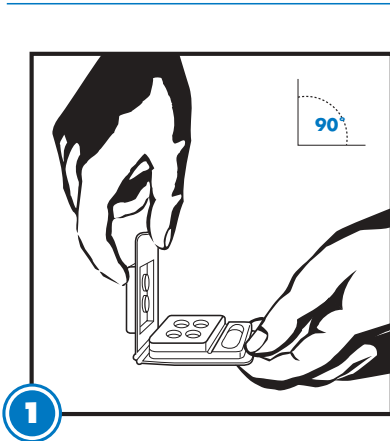

**1** Hold dispenser with lid at 90° to base.

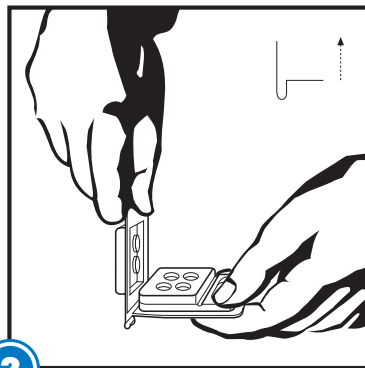

**2** Press base upward to form a "U" with base and lid.

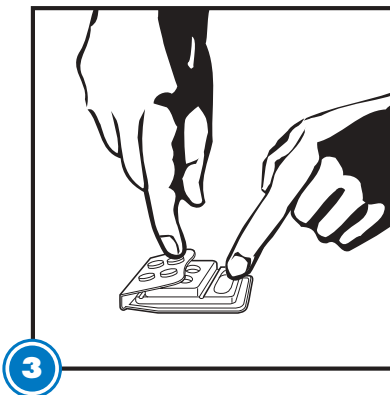

**3** Fold lid over to engage lock to wells.

If lid does not fully lock upon first attempt, adjust hinge as shown in figures 1 - 3.

The Pure Bond Dispenser is designed to hold bonding agents, composites and sealants in four storage wells. The large oval well can also be used for this purpose, and is ideal for storing applicator sponges, cotton balls and finishing burs.

Materials can be placed for extended storage with the lid closed. The light lock lid prevents any light from contaminating agents.

The narrow channels are designed to store applicator brushes prior to, or during procedures.

- Self-contained light lock lid protects against light contamination of photosensitive materials.
- Durable molded resin is compatible with dental composites, bonding agents and primers.
- Disposability helps reduce cross contamination.
- Safe storage of materials before use.

We manufacture a series of quality dental products including the Endoring II, Endoring II Color Value Pack, EndoGel, Pure Buff and Pure Floss Dispenser.



**Jordco, Incorporated**  
 595 NW 167th Avenue, Beaverton, OR 97006 USA  
 800/752-2812 (within USA), 503/531-3904 (outside USA)  
 503/531-3757 FAX  
 www.jordco.com

**Authorized Representative in the EU**  
 AR-MED Ltd.  
 Runnymede Malthouse  
 Egham TW20 9BD  
 United Kingdom