

# ENDORING® II COLOR VALUE PACK

ER2CVP

## FUNCTIONS

New colored Endoring IIs add the dimension of color coding to instrument set-ups. Like the original blue Endoring II, the new colors enable the clinician to directly place, store, measure and clean endodontic instruments within the operating field. Together they reduce the risk of cross contamination and minimize the passing of sharps between dentists and staff. Using the Endoring II helps make endodontic procedures safer and faster.

## TECHNICAL SUMMARY

### ENDORING II CVP:


Assemblies are molded from an autoclavable thermoplastic resin. Assembly consists of six pieces: foam-holder cup, reusable Gelwell® cup attachment, disposable Gelwell cup holder attachment, large, small and petite snap-on finger rings. Note: the foam-holder cup has a molded plastic 30-mm scale for right and left handed users.

### ENDORING METAL RULER:

Sold separately (REF# ERR-s). Reusable Cup: 2ml cup is molded from an autoclavable blue thermoplastic resin and attaches to the Endoring II. Disposable Cups: Single-use 0.65ml cups molded from a clear thermoplastic resin. Disposable cup is placed into the holder and attaches to the Endoring II. (Sold separately)

### ENDORING FOAM:

Single-use open cell yellow and blue foam rubber. Can be steam autoclave processed prior to use. During steam autoclave processing, endodontic instruments can be inserted into the foam.

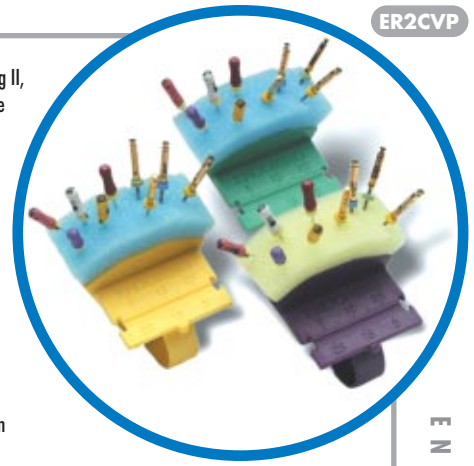

## PACKAGING

All Endoring kits and inserts are packaged in poly bags and bar coded to HIBC convention.

## PRODUCT CODES

REF#	ITEM DESCRIPTION	REF#	ITEM DESCRIPTION
ERK2CVP-s	Endoring II Color Value Pack: 3 Endoring II assemblies (yellow, green, violet). (metal ruler not included)	ERFB-s	Foam Inserts Blue Pack: Single pack of 48 inserts (48 blue)
ERR-s	Single Endoring metal ruler (standard)	ERFY-s	Foam Inserts Yellow Pack: Single pack of 48 inserts (48 yellow)
ERRFL-s	Single Endoring metal ruler (left handed no-stop lock)	ERK2CVP	Case of Endoring II Color Value Packs (25/case)
ERRFR-s	Single Endoring metal ruler (right handed no-stop lock)	ER2	Case of Endoring II (12/case)
ERG-s	Single package of 48 single use Gelwells	ERG	Case of Gelwell cups (24/case)
ERF-s	Foam Inserts Combo Pack: Single pack of 48 inserts: (24 blue and 24 yellow)	ERF	Case of Endoring Foam Inserts Combo (48 blue/yellow packs per case)
		ERR	Case of Endoring Rulers (12/case)

ENDODONTIC HAND INSTRUMENT & ORGANIZING TOOL

ENDORING CVP

COMPARISON GUIDE

FEATURES	ENDORING II	ENDORULERS	ENDO-CUSHIONS	MEASURING GAUGE
Autoclavable	YES	YES/NO	NO	YES
Rubber stop loading ports	YES	NO	NO	NO
Reduces instrument sticks	YES	NO	NO	NO
Disposable foam inserts	YES	NO	YES	NO
Cleans files during use	YES	NO	YES	NO
Reusable or disposable lubricating gel holder	YES	NO	NO	NO
Finger mounted	YES	YES	NO	NO
Direct operator control of all files	YES	NO	NO	NO
Right-left 30mm scale	YES	NO	NO	NO
Easy calibration of curved instruments	YES	NO	NO	NO
Faceted foam surface to separate files	YES	NO	NO	NO
Detachable anodized laser etched ruler	YES	NO	NO	NO
Frees assistant for other duties	YES	NO	NO	NO



800/752.2812  
(Inside the USA)

503/531.3904  
(Outside the USA)

503/531.3757 FAX

www.jordco.com

FEATURES & BENEFITS

### Three Endoring Colors

Three new colors allow the operator to color code each procedure. The rings are sold in an economical 3-pack and contain one green, yellow and violet Endoring II. Each one is the same efficient design as the original blue Endoring II.

### New Foam Color

New yellow color provides an additional option for color coding instrument set-ups. Each bag of 48 refills contains 24 blue inserts and 24 yellow inserts.

### Hand-held Instrument

Allows direct operator control of hand and rotary instruments. Eliminates time consuming passing of dangerous files from assistant or bracket tray. Minimizes eyestrain and loss of attention due to searching for files, etc. Saves several minutes each procedure.

### Held on Index Finger

Frees both hands. Allows operator to hold mouth mirror in one hand while selecting and operating files or instruments with the opposite hand. Includes three ring sizes to fit the majority of fingers. A new extra small ring has been added for the petite hand.

### Standard Right-Left 30mm Scale

The newly designed Endoring ruler is easier to read. It has a recessed calibration tract and improved rubber stop locks. These features enable the operator to position files, reamers, paper points, gutta percha and other endodontic instruments into the tract and rapidly calibrate them to working length. The scale is designed for both right and left handed users.

### Detachable Laser Etched Metal Ruler

Autoclavable metal ruler with a precise laser etched right or left-handed 30mm scale. Includes 1/2 mm calibration markings on the lower half of the scale. The metal ruler also includes a secondary scale to measure files up to 37-mm in length. Includes a calibration tract for easy positioning of instruments. Ruler can be easily detached for cleaning and processing.

### Easy Calibration of Curved Instruments

Curved files can be easily positioned into the calibrating tract within the standard or metal ruler and temporarily straightened with slight pressure. After the stop is adjusted, the file is removed and allowed to reassume its curved shape.



Steam autoclave only prior to use. **DO NOT REUSE FOAM INSERT.** Autoclaving **will not** remove bioburden left behind from prior usage. Do not exceed temperature of 275°F (135°C).

**Warning:** Do not place any components into Surgical Milk or instrument dips prior to autoclaving.

U.S. Patents 5368482, 6036490, Patent Pending  
Made in USA

