



FUNCTIONS

The Endoring FileCaddy provides a unique bulk storage solution for all endodontic files. The clinician places multiple files securely into the special foam insert, closes the lid and steam autoclaves the entire FileCaddy. Single handed operation allows easy opening of the protective lid to access the processed files at chair side. Files are easily identified, selected and transferred to the companion Jordco Endoring II for the current endodontic procedure. The FileCaddy disassembles easily to allow replacement of the foam insert.

TECHNICAL SUMMARY

FILECADDY CASE:

The Endoring FileCaddy case is constructed of a special glass reinforced thermoplastic resin that can be safely steam autoclaved (275°F/135°C). There are three molded components that make up the FileCaddy case; a molded base, retainer and lid. All the components are engineered to easily and reliably assemble. A specially designed retainer locking mechanism securely holds the foam insert onto the base. The color of the case is a light beige tone. Size (assembled FileCaddy): Approximately 4 5/8" w x 4 5/8" d x 3" h.

FILECADDY FOAM INSERT:

The disposable EFC Foam Insert is constructed of Jordco's proprietary open cell foam that is also used in our Endoring Foam Inserts. The EFC Foam Insert and case can be steam autoclave processed (275°F/135°C). The foam inserts are available in blue or yellow color.

PACKAGING

The Endoring FileCaddy is packaged in a corrugated carton with one Foam Insert Combo package (6 yellow and 6 blue). Foam Inserts are packaged in a clear poly bag.

PRODUCT CODES

REF#

ITEM DESCRIPTION

EFC-s	Single Endoring FileCaddy Kit: 1 FileCaddy and 12 single use foam inserts (6 yellow and 6 blue)
EFCF-s	Single Endoring FileCaddy Foam Inserts Combo pack: 12 foam inserts per bag (6 blue and 6 yellow)
EFCFB-s	Endoring FileCaddy Foam Inserts Blue: 12 blue foam inserts
EFCFY-s	Endoring FileCaddy Foam Inserts Yellow: 12 yellow foam inserts
EFC	Case of Endoring FileCaddy Kits: 12 kits per case
EFCF	Case of Endoring FileCaddy Foam Inserts Combo: 24 Single packs per case (blue/yellow)
EFCFB	Case of Endoring File Caddy Foam Inserts Blue: 24 Single packs per case (blue only)
EFCFY	Case of Endoring File Caddy Foam Inserts Yellow: 24 Single packs per case (yellow only)



800/752.2812
(Inside the USA)

503/531.3904
(Outside the USA)

503/531.3757 FAX

www.jordco.com

COMPARISON GUIDE

FEATURES	FILECADDY	ENDO CLEAN STAND (DENTSPLY)	METAL ENDO BOXES (VARIOUS MFG.)
Replaceable foam insert	YES	YES	NO
Attached locking lid	YES	NO	NO (HAS LID)
4 bulk storage compartments	YES	SINGLE	MULTI SINGLE HOLE
One-hand operation	YES	NO	NO
Use at chair side	YES	NO	YES
Steam Autoclavable	YES	YES	YES
Spill proof foam insert	YES	YES	NO
Autoclave bag required	NO	NO	NO

FEATURES & BENEFITS

Locking Lid	Files are held securely in place between procedures and will not spill out if the FileCaddy is dropped or turned upside down.
Autoclavable and Durable	The FileCaddy is constructed of a medical grade resin that can withstand repeated steam autoclaving (275°F/135°C). The resin is glass reinforced for added durability and longevity.
4 Bulk Storage Compartments	The FileCaddy is divided into 4 convenient bulk storage compartments to help make file organization easy. The separate compartments simplify organization and retrieval of instruments. Over 100 instruments can be safely stored in the File Caddy for steam autoclaving.
Disposable Foam Inserts	The foam inserts are disposable to reduce cross contamination*.
Two Colors of Foam Inserts	Two colors are helpful to organize/color code file setups. Inserts can be purchased in three different refill packs: Combo (6 Blue/6Yellow), Blue (12 blue) or Yellow (12 yellow) packs.
Secure File Loading	Foam inserts eliminates the risk of spilling files- minimizes lost files.
One-Hand Operation	One hand operation allows for easy opening of the locking lid to access files and contents.
Simple to Use	Bulk load endodontic files, close locking lid and steam autoclave.
Save Files	Steam autoclave only those files that are actually used in the procedure. No unnecessary steam autoclaving of unused files.

* The Foam Insert should be replaced every two autoclaving cycles to prevent discoloration of the EFC resin case. Should discoloration occur, simply disassemble the EFC and soak the components in sodium hypochlorite over night to remove discoloration.



Steam autoclave only prior to use. **DO NOT REUSE FOAM INSERT.** Autoclaving **will not** remove bioburden left behind from prior usage. Do not exceed temperature of 275°F (135°C).

Warning: Do not place any components into Surgical Milk or instrument dips prior to autoclaving.

U.S. Patents Pending
Made in USA

